XLT4 Supplement

Instructions For Use
ENGLISH

Selective supplement for detection of non-typhi *Salmonella* spp.

DESCRIPTION

A liquid supplement for use with XLT4 (Xylose Lysine Tergitol 4) Agar Base (Ref. 610092 / 620092) for the detection and isolation of non-typhi *Salmonella* from clinical and nonclinical samples.

COMPOSITION

Ingredient	Content per bottle
Niaproof® 4 (formerly known as Tergitol 4)	50 ml

Each bottle is sufficient for 10 litres of medium.

METHOD PRINCIPLE

XLT4 Supplement is an anionic surfactant, which is added to the medium to suppress the growth of many gram-negative and gram-positive bacteria.

DIRECTIONS

1. Suspend 59.0 g of XLT4 Agar Base in 1 litre of purified water.
2. Add 4.6 ml of XLT4 Supplement.
3. Bring to boil and shake to completely dissolve the medium. DO NOT OVERHEAT OR AUTOCLAVE.
4. Cool to 45-50°C before distributing into sterile Petri dishes.

TECHNIQUE AND INTERPRETATION OF THE RESULTS

Refer to the technical sheet of the medium being prepared.

STORAGE

10-25°C in its original packaging. Keep away from sources of heat and avoid excessive changes of temperature. Use until the expiry date indicated on the label. Eliminate without using if there are signs of deterioration.

SHELF LIFE

2 years.

QUALITY CONTROL

Appearance: Clear, colourless to pale yellow.

Microbiological Performance:

Control strains

Salmonella Typhimurium WDCM 00031 (ATCC® 14028, NCTC 12023)

Escherichia coli WDCM 00013 (ATCC® 25922, NCTC 12241)

Please refer to the actual batch related Certificate of Analysis (CoA).

WARNING AND PRECAUTIONS

For In Vitro Diagnostic use. For professional use only. Operators must be trained and have certain experience. Please read the instructions carefully before using this product. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this document.

Consult the Safety Data Sheet (SDS) for information regarding hazards and safe handling practices.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to national and local regulations in force.

BIBLIOGRAPHY

See the references at the end of this document.

TABLE OF SYMBOLS

See the table of symbols at the end of this document.

Product	Packaging	Ref.
XLT4 Supplement	2 x 50 ml	80010
XLT4 Supplement	4 x 50 ml	80410

This IFU document and the SDS are available from the online Support Center:

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liofilchem.com/ifu-sds

BIBLIOGRAPHY / BIBLIOGRAFIA

1. EN ISO 11133:2014+Amd1:2018. Microbiology of food, animal feed and water – Preparation, production, storage and performance testing of culture media.
2. Dusch, H. and Altwegg, M. (1995) Evaluation of five new plating media for the isolation of Salmonella species. *Journal of Clinical Microbiology*. 33. No.4. 802-804.
3. Miller, R.G., Tate, C.R., Mallinson, E.T. and Scherrer, J.A. (1992) Erratum. Xylose-Lysine- Tergitol 4: An improved selective agar for the isolation of Salmonella. *Poultry Science* 71. 398.
4. Tate, C.R., Miller, R.G. and Mallinson, E.T. (1992) Evaluation of two isolation and non-isolation methods for detecting naturally occurring salmonellae from broiler flock environmental drag- swab samples. *J. Food Prot.* 55. 964-967.
5. Miller, R.G., Tate, C.R., Mallinson, E.T. and Scherrer, J.A. (1991) Xylose-Lysine-Tergitol 4: An improved selective agar for the isolation of Salmonella. *Poultry Science* 70. 2429-2432.
6. Miller, R.G. and Tate, C.R. (1990) A highly selective plating medium for the isolation of Salmonella. *The Maryland Poultryman*, April: 2-7.

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